



Magneto
Transduction Therapy
Texas Wellness

Dear Valued Customer,

Welcome to the future of wellness with the Magneto Transduction Therapy device. For optimal results and a seamless experience, please follow these essential guidelines:

- ✓ Read this guide thoroughly and keep it for future reference.
- ✓ Assemble and operate the machine as instructed.
- ✓ Do not modify or replace any parts of the machine.
- ✓ Only authorized personnel should open the machine.

If you need assistance, our support team is here to help.

Discover the Power of Magneto Transduction Therapy Therapy

Unlock your body's healing potential with the revolutionary Magneto Transduction Therapy therapy. This cutting-edge technology enhances cellular charge using High Tesla pulsed electric magnetic fields.

Key Benefits:

- Pain Relief: Significantly reduce pain without discomfort.
- Enhanced Circulation: Improve both superficial and deep blood flow.
- Tissue Regeneration: Promote rapid tissue repair.
- Lymphatic Drainage: Stimulate venous and lymphatic drainage.
- Metabolic Balance: Restore and maintain your body's metabolic equilibrium.

Our device penetrates up 5 cm to 18 cm under the skin, targeting bones, fascia, ligaments, and muscles for comprehensive healing. Helpful for treating bone fractures, chronic pain, sports injuries, and more.

Indications for Therapy:

- Muscular Pathologies: Contractures, muscle tears, bruises, myalgia and swelling.
- Joint Issues: Injuries, osteoarticular sprains, wear and tear (shoulder, hip, knee, ankle).
- Arm Pathologies: Epicondylitis, tendinitis, carpal tunnel syndrome.
- Spinal Pathologies: Cervical, thoracic, and lumbar issues.
- Tendon Issues: Achilles tendon and patellar ligament inflammation and damage.
- Shoulder Issues: Tendonitis and calcification in the shoulder joint, acute and chronic edema.

Easy Device Operation



Step-by-Step Guide:

1. Setup:

- ✓ Insert the handle and power cord into their respective sockets on the back of the device.
- ✓ Turn on the power switch; the device will start immediately.
- ✓ Adjust the support arm to your desired angle and direction.

2. System Interface:

- ✓ After booting up, click on the boot interface to enter the body part selection interface.
- ✓ Choose the body part you wish to treat (e.g., shoulder).

3. Adjust Settings:

- ✓ Energy Intensity: Set the intensity from 1 (weak) to 12 (strong). Start at a lower level like 3 and increase based on client feedback.
- ✓ Time: Set the treatment time, typically 15-20 minutes per session.
- ✓ Frequency Intensity: Adjust the frequency according to the client's needs.

4. Start Therapy: Click “START” to begin the session.

Figure 1: Boot Interface

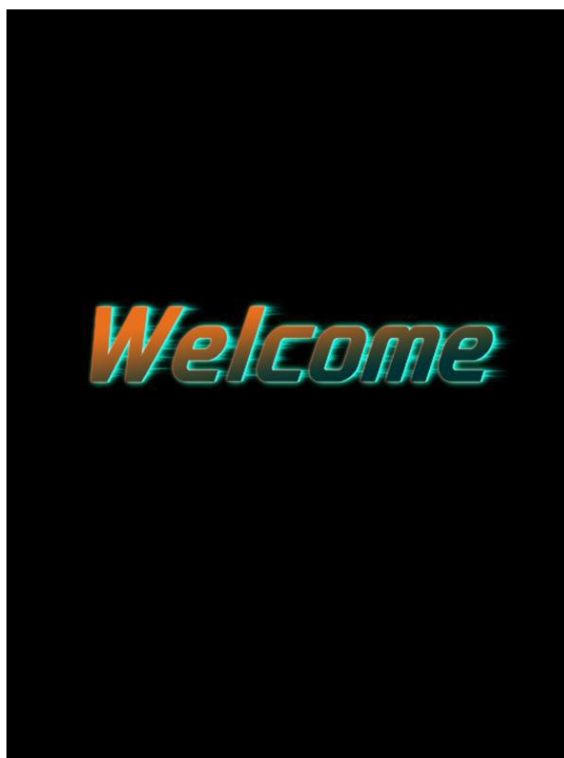


Figure 2: Body Part Selection



Dual Mode Treatment

TEXAS WELLNESS

TIME ENERGY

Increase time — (+)

Decrease time — (-)

Increase energy — (+)

Decrease energy — (-)

Body Part: Cervical Vertebra

F1 — [Slider] — (+) — [Hz]

F2 — [Slider] — (+) — [Hz]

F3 — [Slider] — (+) — [Hz]

ST MT PAUSE <<

ST mode is Auto mode- Preset parameters based on 10 different treatment areas.

MT mode is Manual mode- Adjust parameters based on actual situation of patients.

One treatment may divide into 3 different periods.
Frequency for the 3 different periods

Back to last page

Start or stop the operation

MT Mode

- High-frequency superconducting for deep analgesia and bone healing.

ST Mode

- Low-frequency repair for superficial muscle issues and accelerated recovery.

Advantages:

- ✓ **Deep Penetration:** The therapy ring delivers high-energy magnetic pulses to muscles and nerve tissues, working without direct skin contact. Therapy can be given with clothes on
- ✓ **Comfortable Experience:** Each session lasts 15-20 minutes, with 6-8 sessions recommended for full recovery.
- ✓ **Advanced Technology:** Our device features up to ultra-fast 1KHZ frequency, 4 Tesla energy, and has innovative air-cooling technology for prolonged sessions.
- ✓ **Privacy:** Magneto Transduction Therapy waves can penetrate clothing, eliminating the need to undress during therapy sessions.

Safety and Precautions

Contraindications:

- ✓ Presence of metal nails in the body
- ✓ Unhealed ulcerated infected wounds
- ✓ Avoid therapy near the brain and heart
- ✓ Not suitable for children and certain special groups

Forbidden Areas:

- ✓ Eyes and surrounding area, myocardium, spinal cord, gonads, kidneys, and liver
- ✓ Conditions like coagulation disorders, thrombosis, malignant diseases, and polyneuropathy
- ✓ Areas near large nerve bundles and blood vessels

Possible Side Effects:

- ✓ Temporary erythema or swelling
- ✓ Temporary loss of sensation or itching
- ✓ Skin damage in cases of previous corticoid therapy

Embrace a New Era of Health and Wellness

Magneto Transduction Therapy therapy is a breakthrough in non-invasive therapy, good for muscle strengthening, pain management, rehabilitation, and more. This innovative device stimulates deeper muscles and nerves, inducing muscle contraction and neuromodulation for positive therapeutic effects. Experience muscle strengthening, reduced pain and swelling, and improved range of motion. Enhance your overall health by boosting cell metabolism, increasing oxygen absorption, and supporting a healthier immune system.

PRODUCT APPLICATION



Join the many who have transformed their lives with Magneto Transduction Therapy therapy. Welcome to a more comfortable therapy experience and better outcomes.

Texas Wellness - Magneto Transduction Therapy

Your pathway to advanced healing and wellness.